

WOW!



MIDWEST®
STYLUS™ ATC
ANYTHING ELSE IS ORDINARY

MIDWEST[®] STYLUS[™] ATC

The Midwest Stylus ATC represents the first significant change in handpiece technology in a very long time. It has Speed-Sensing Intelligence (SSI) that optimizes power to virtually eliminate stalling even under high cutting loads – thus providing unmatched cutting efficiency for faster removal of material. It also features a Superior Turbine Suspension (STS) that dramatically reduces bur deflection and chatter to maximize control and precision.

SPEED-SENSING INTELLIGENCE (SSI)

- Incorporates a sensor in the coupler that sends a signal approximately 750 times per second to the electronic brain to adjust bur speed according to load. It knows when to deliver more power!
- Delivers smooth, consistent power and control for unmatched cutting efficiency and faster removal of material.
- Automatically optimizes power to virtually eliminate stalling and the need to make frequent adjustments.
- Adjusts speed when the bur is not under load to minimize wear on handpiece bearings.

OUTSTANDING PRECISION, ACCURACY & CONTROL

SUPERIOR TURBINE SUSPENSION (STS)

- Allows the handpiece to operate at speeds of 330,000 RPM under load without noticeable bur deflection or chattering for better control and precision.
- Creates a greater, more consistent transference of power from the handpiece to the bur for maximized cutting efficiency for procedures like crown removal (PFM, porcelain and metal), amalgam removal, etc.
- Helps create the degree of precision the Dentist needs time after time
- Smooth precision and control for delicate procedures such as margin refinement, veneer preparations and other fine restorative procedures.

SUPERIOR PERFORMANCE



- Most powerful air-driven handpiece.
- High power at constant speed for fast, efficient removal of most materials.
- Performance does not drop off under load like conventional air-driven handpieces. See performance graph and power table on brochure for proof of superiority.

USER-FRIENDLY DESIGN

- A swivel with exceptional rotation capabilities for freedom of movement.
- All-day ergonomic comfort – significantly lighter than electric handpieces.
- Exceptional visibility, access and maneuverability with Mini- or Mid-Sized-Head.
- Accepts both standard- and short-shank burs for great convenience.
- Enhanced access to posterior surfaces with short-shank bur and small head size.
- A low pitch and tone for a more relaxed patient and Dentist.

MIDWEST®
STYLUS™ ATC
 ANYTHING ELSE IS ORDINARY



Midwest® Stylus™ ATC PLUS Starter Package 	884100	ATC 890 – Three Mid-Sized-Head Handpieces, one Control Source, one Power Supply, one Coupler and Tubing, Installation Kit and Maintenance Kit	\$4,400.00
	884200	ATC 990 – Three Mini-Head Handpieces, one Control Source, one Power Supply, one Coupler and Tubing, Installation Kit and Maintenance Kit	\$4,400.00
Midwest® Stylus™ ATC Starter Package 	883100	ATC 890 – Two Mid-Sized-Head Handpieces, one Control Source, one Power Supply, one Coupler and Tubing, Installation Kit and Maintenance Kit	\$3,509.00
	883200	ATC 990 – Two Mini-Head Handpieces, one Control Source, one Power Supply, one Coupler and Tubing, Installation Kit and Maintenance Kit	\$3,509.00
Midwest® Stylus™ ATC Handpiece	881300	ATC 890 – One Mid-Sized-Head Handpiece	\$1,399.00
	882300	ATC 990 – One Mini-Head Handpiece	\$1,399.00

MIDWEST® STYLUS™ ATC MAINTENANCE PRODUCTS

Item #	Product	MSRP
380080	Midwest® Plus™ Aerosol Spray Cleaner and Lubricant in a convenient aerosol spray (nozzle sold separately)	\$52.49
380130	Midwest® Plus™ Handpiece Lubricant, 2 oz. dropper	\$39.49
380150	Midwest® Spray Guards	\$31.99
380800	Midwest® Handpiece Air Station	\$344.99
880080	Midwest® Stylus™ ATC Nozzle for Midwest® Plus™ Aerosol Spray	\$14.49
880090	Midwest® Stylus™ ATC Maintenance Coupler	\$97.49

MIDWEST® STYLUS™ ATC REPLACEMENT PARTS

Item #	Product	MSRP
880060	Midwest® Stylus™ ATC Installation Hardware Kit	\$37.49
880070	Midwest® Stylus™ ATC O-ring Replacement Kit	\$9.49
880200	Midwest® Stylus™ ATC Replacement Power Supply (Adapter and Cord)	\$190.99
880500	Midwest® Stylus™ ATC Replacement Coupler / Tubing and Installation Hardware	\$500.00
880600	Midwest® Stylus™ ATC Replacement Controller Unit, Coupler / Tubing and Installation Hardware	\$839.00
880040	Replacement Bulbs for Midwest® Stylus™ ATC	\$66.99
880700	Midwest® Stylus™ ATC Control Source Kit (Control Source, Coupler/Tubing, Installation Hardware Kit, Power Supply)	\$1,500.00
651248R	Remote Intensity Control	\$65.49

MIDWEST[®] STYLUS[™] ATC CONTACT INFORMATION

Customer Service 800-989-8825

Technical Customer Service/Air Repair 800-800-7202

Installation Technician Helpline 877-794-8357

www.StylusATC.com

MIDWEST[®]
STYLUS[™] ATC
ANYTHING ELSE IS ORDINARY

90674-Rev-1