

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



WOW!  
IS NOW.

## The Midwest® Stylus™ ATC Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

DENTSPLY  
PROFESSIONAL

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS

▼

A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

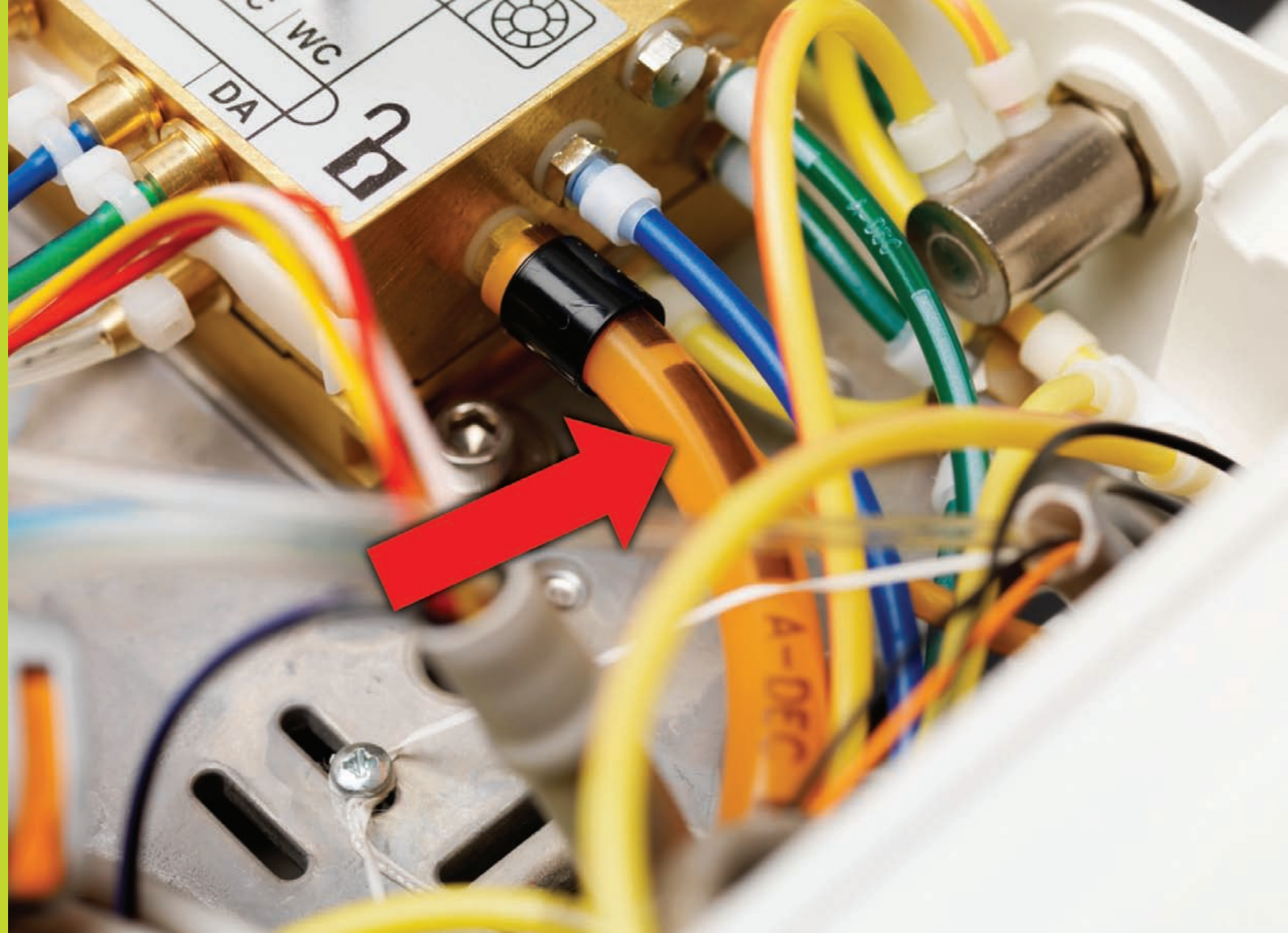

## A-dec® 334

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## A-dec® 334

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Remove the Main Air Tee from the white tube and use the one supplied in the install kit.

Place the foam pad on the mounting bracket to prevent it from sliding around.

You will not need the oil cup provided in the starter kit. Connect to the A-dec Oil Collector.

## A-dec® 334

Connecting the Controller Main Air | Mounting the Controller | Installation Tips



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

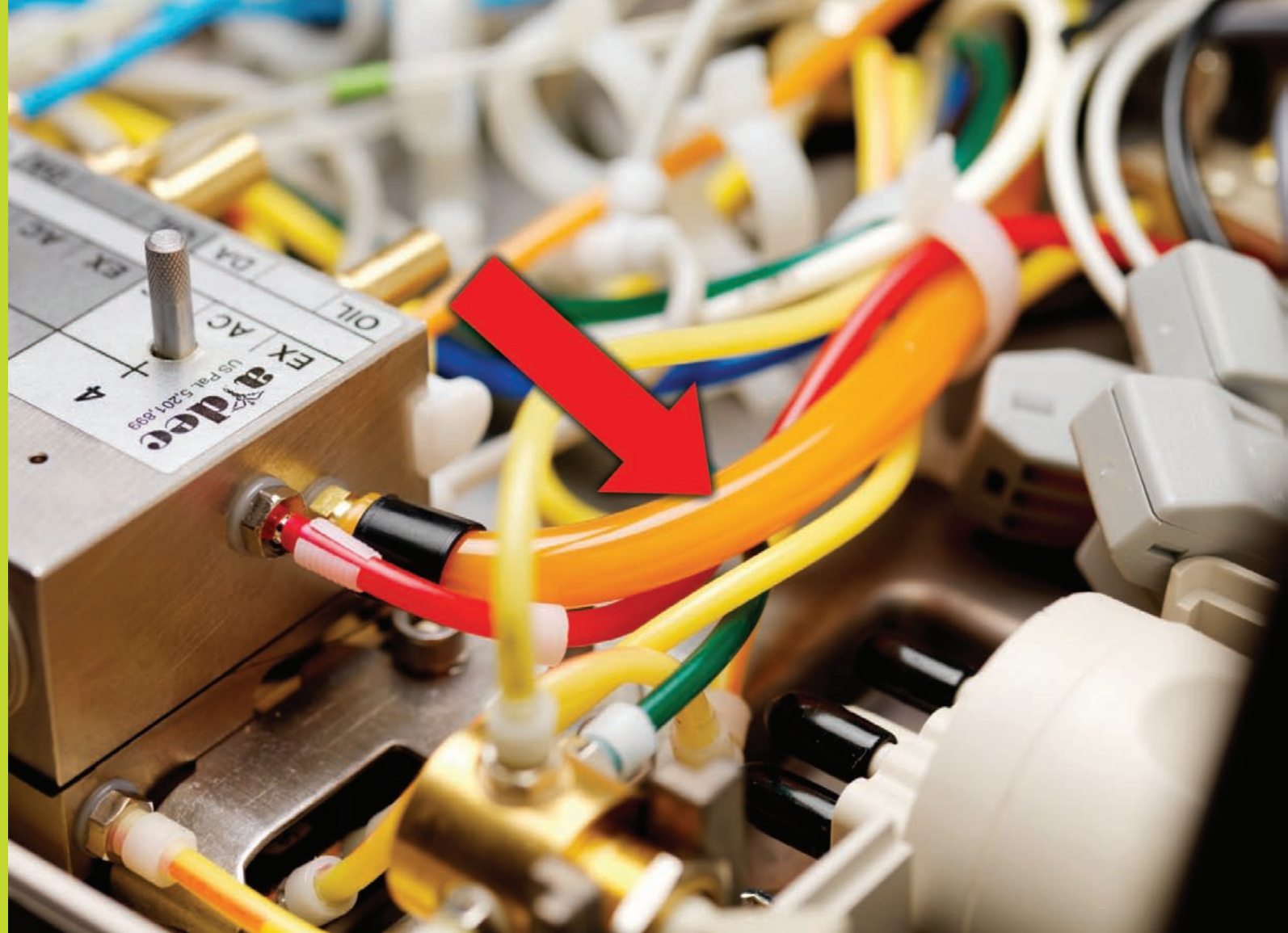

## A-dec® 532

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## A-dec® 532

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Remove the Main Air Tee from the white tube and use the one supplied in the install kit.

Place the foam pad on the mounting bracket to prevent it from sliding around.

You will not need the oil cup provided in the starter kit. Connect to the A-dec Oil Collector.

## A-dec® 532

Connecting the Controller Main Air | Mounting the Controller | Installation Tips



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

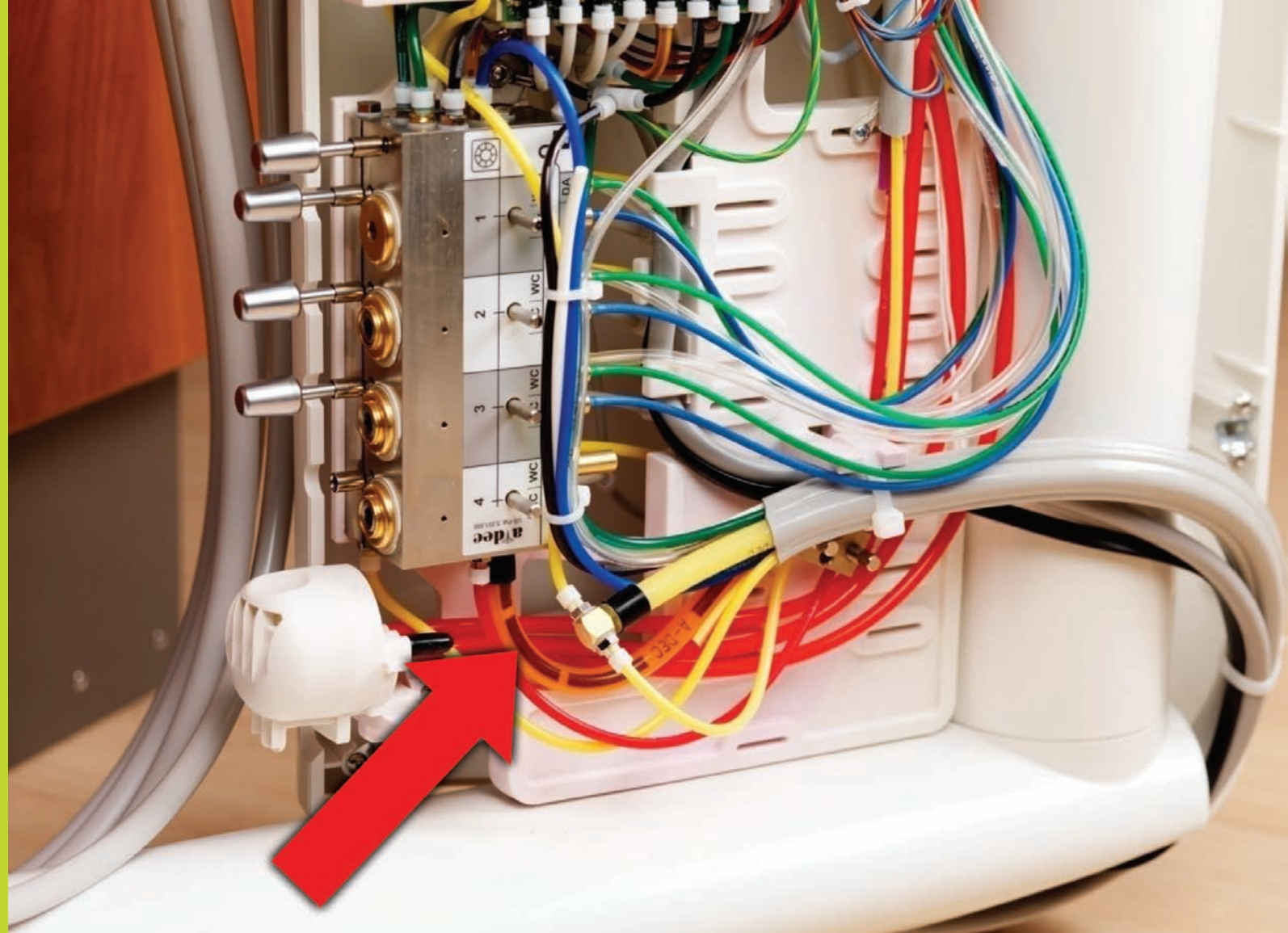

## A-dec® 541

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

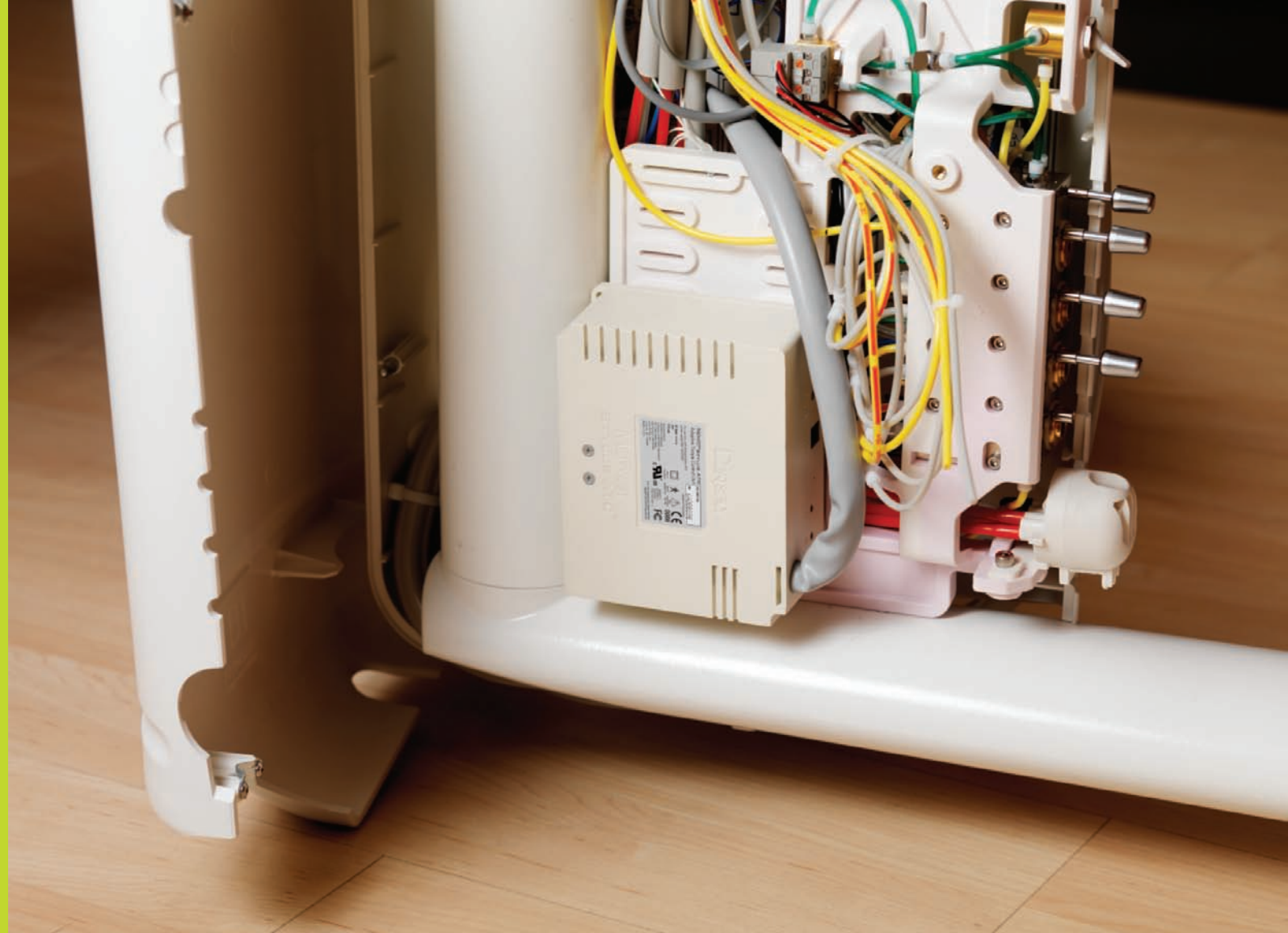

## A-dec® 541

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Remove the Main Air Tee from the white tube and use the one supplied in the install kit.

The controller can be screwed into the tray on the opposite side of the control block and the tubing fed around.

You will not need the oil cup provided in the starter kit. Connect to the A-dec Oil Collector.

## A-dec® 541

Connecting the Controller Main Air | Mounting the Controller | Installation Tips



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS

▼

A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

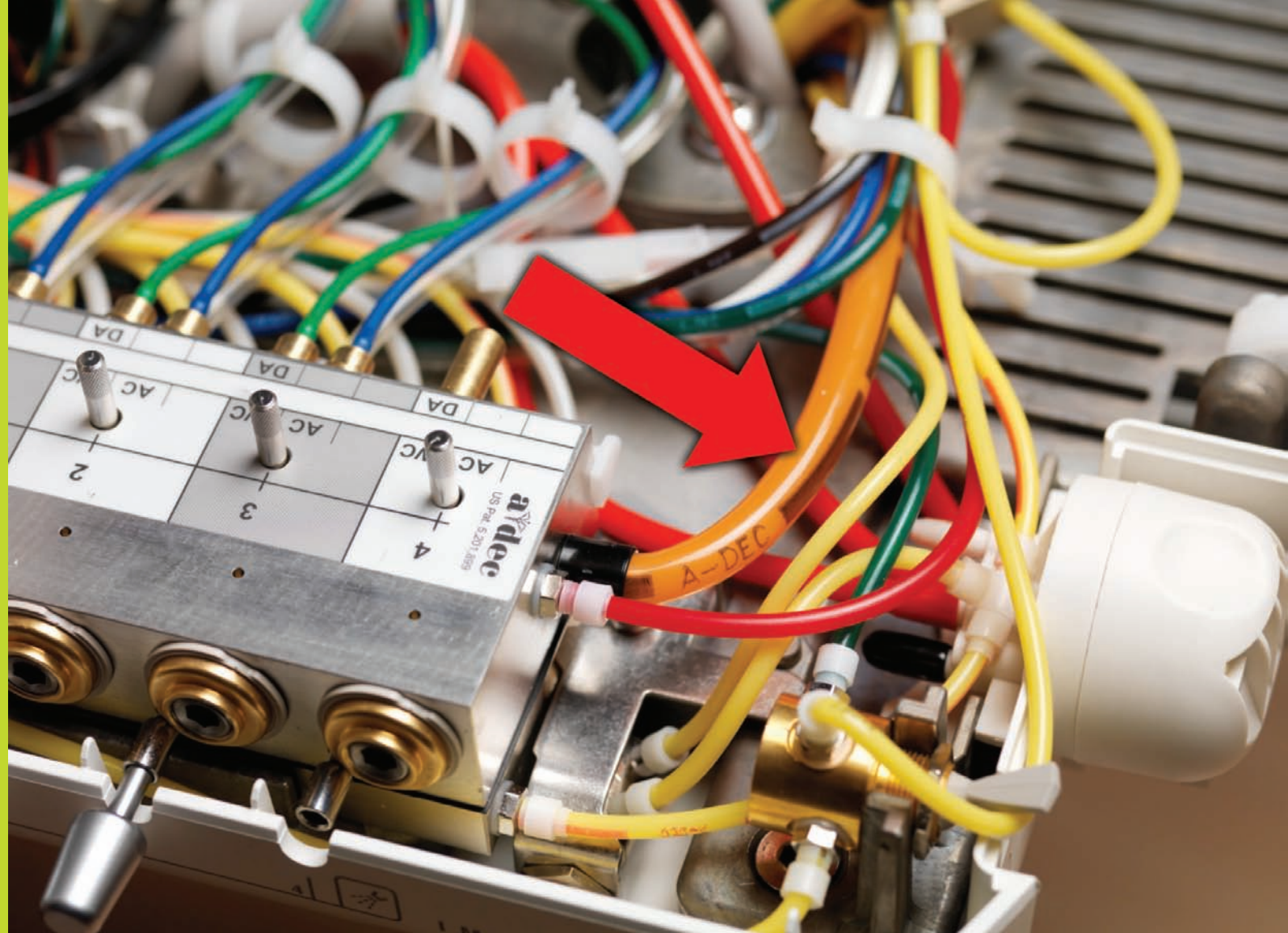

## A-dec® 542

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

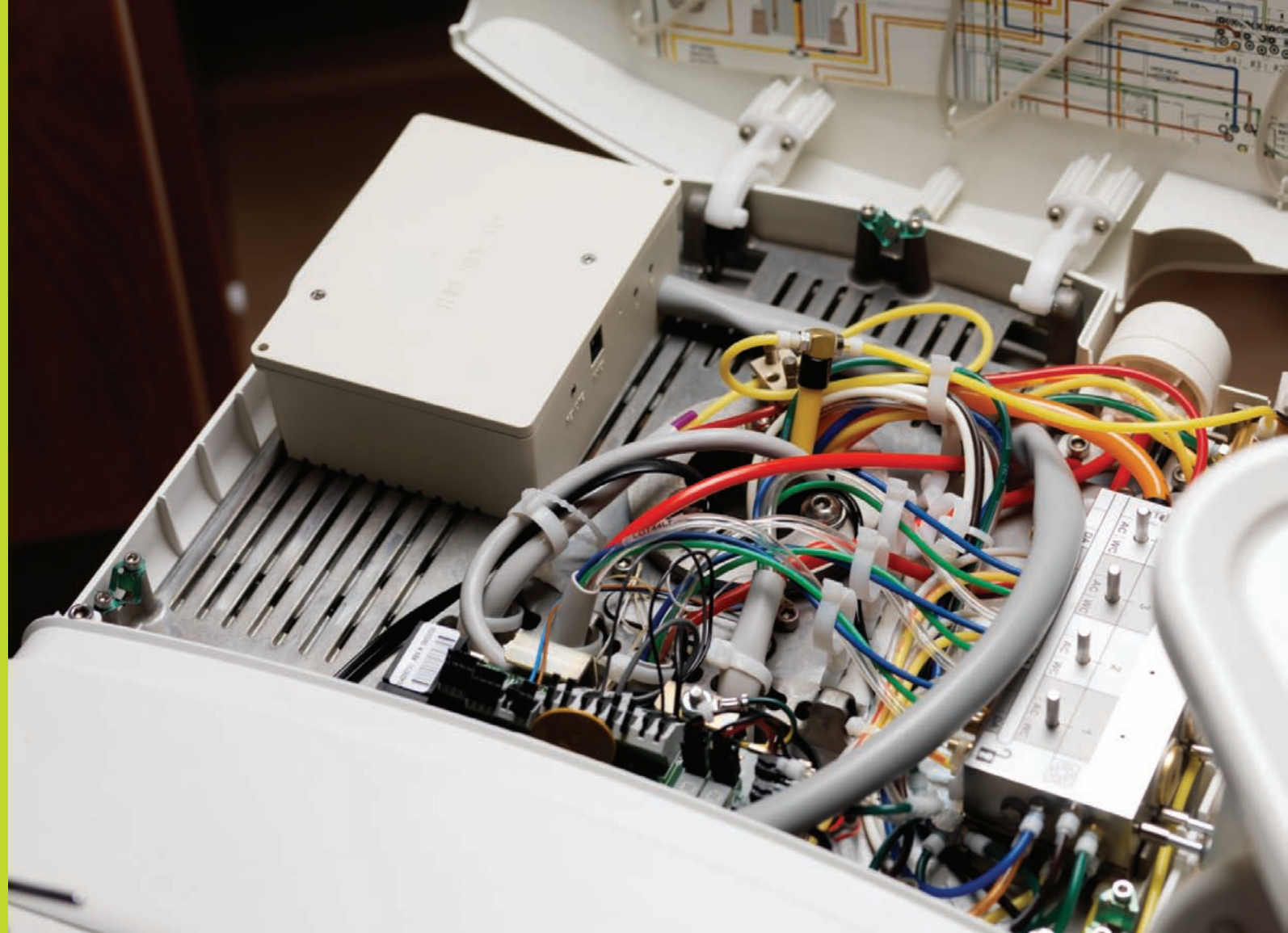

## A-dec® 542

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Remove the Main Air Tee from the white tube and use the one supplied in the install kit.

The unit can be placed inside the delivery head.

You will not need the oil cup provided in the starter kit. Connect to the A-dec Oil Collector.

## A-dec® 542

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

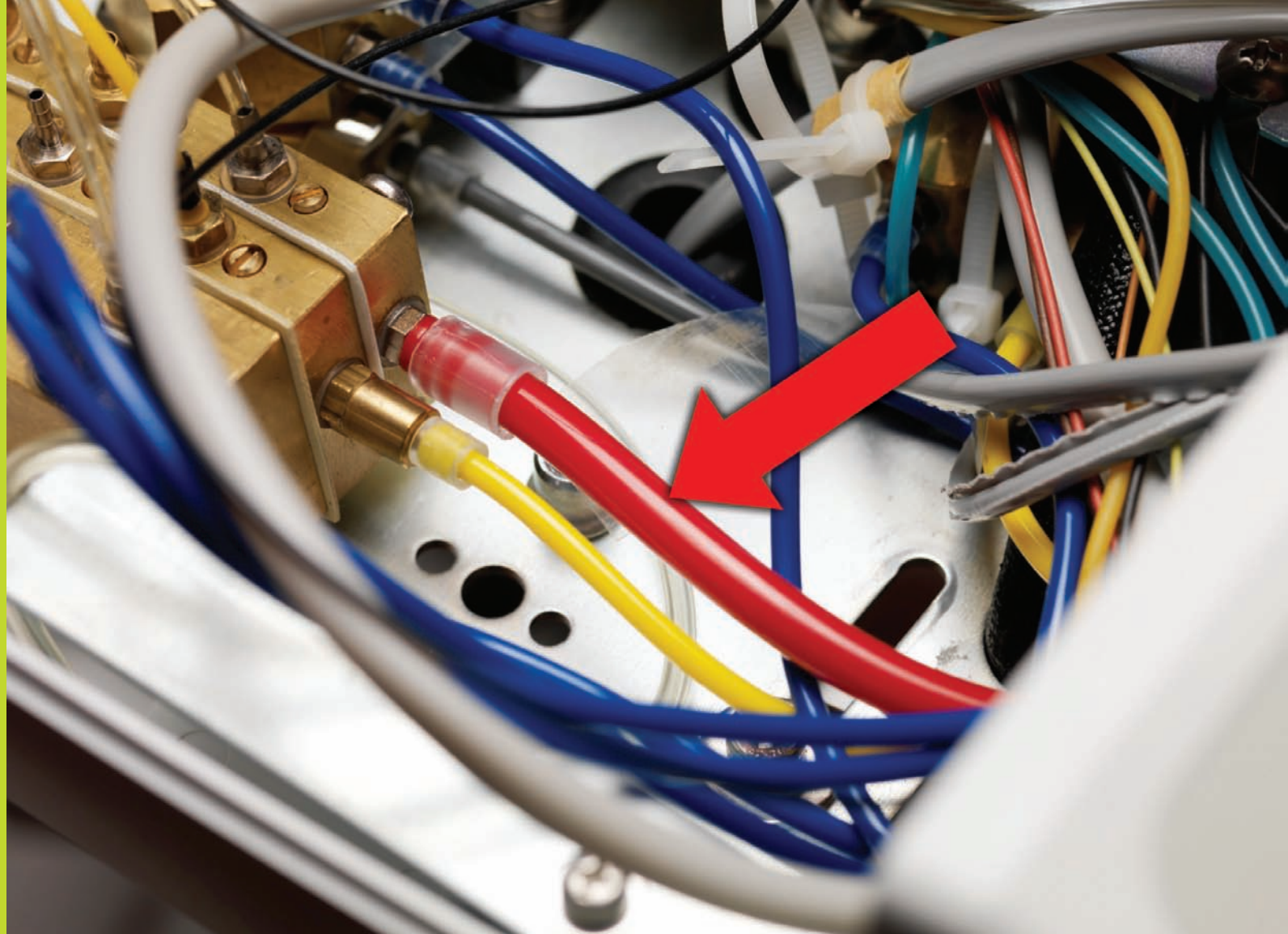

## Belmont Quolis® 5000

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Belmont Quolis® 5000

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Remove the Main Air Tee from the white tube and use the one supplied in the install kit.

Place the foam pad on the mounting bracket to prevent it from sliding around on the boom. Make sure you leave enough slack to allow for the delivery head to rotate.

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the delivery head.

## Belmont Quolis® 5000

Connecting the Controller Main Air | Mounting the Controller | Installation Tips



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Biotec N7125

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Biotec N7125

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Bolt the controller to the panel or use the self tapping screws provided in the install kit. Velcro can be used but surfaces must be free of oil.

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the pull-out panel.

## Biotec N7125

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

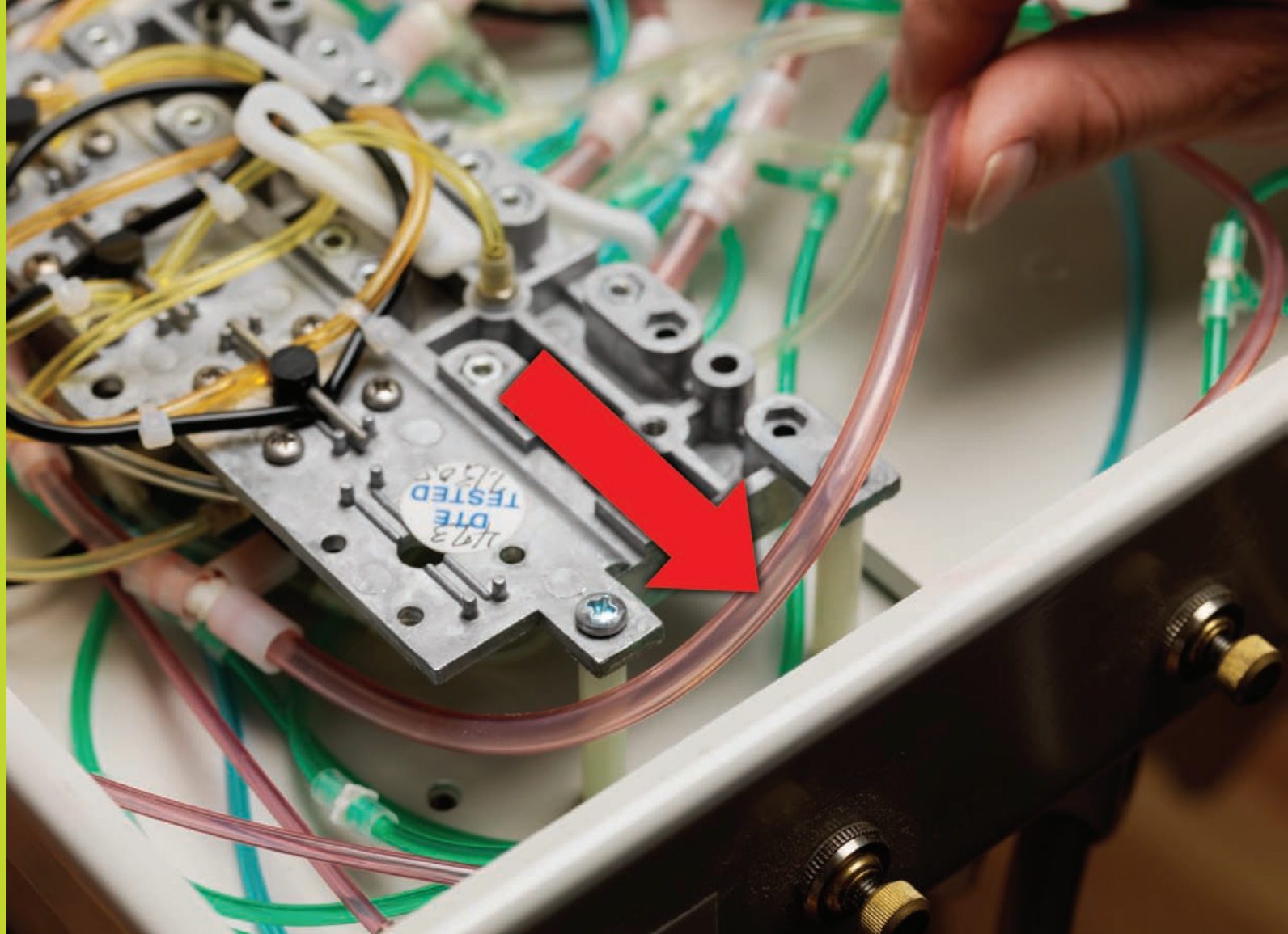

## DentalEZ® Galaxy®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL




MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS

▼

- A-dec® 334
- A-dec® 532
- A-dec® 541
- A-dec® 542
- Belmont Quolis® 5000
- Biotec N7125
- DentalEZ® Galaxy®
- Forest 4485
- Marus MAX100A
- Midmark 153600-8
- Pelton and Crane FWS
- Simplicity™ from DentalEZ®

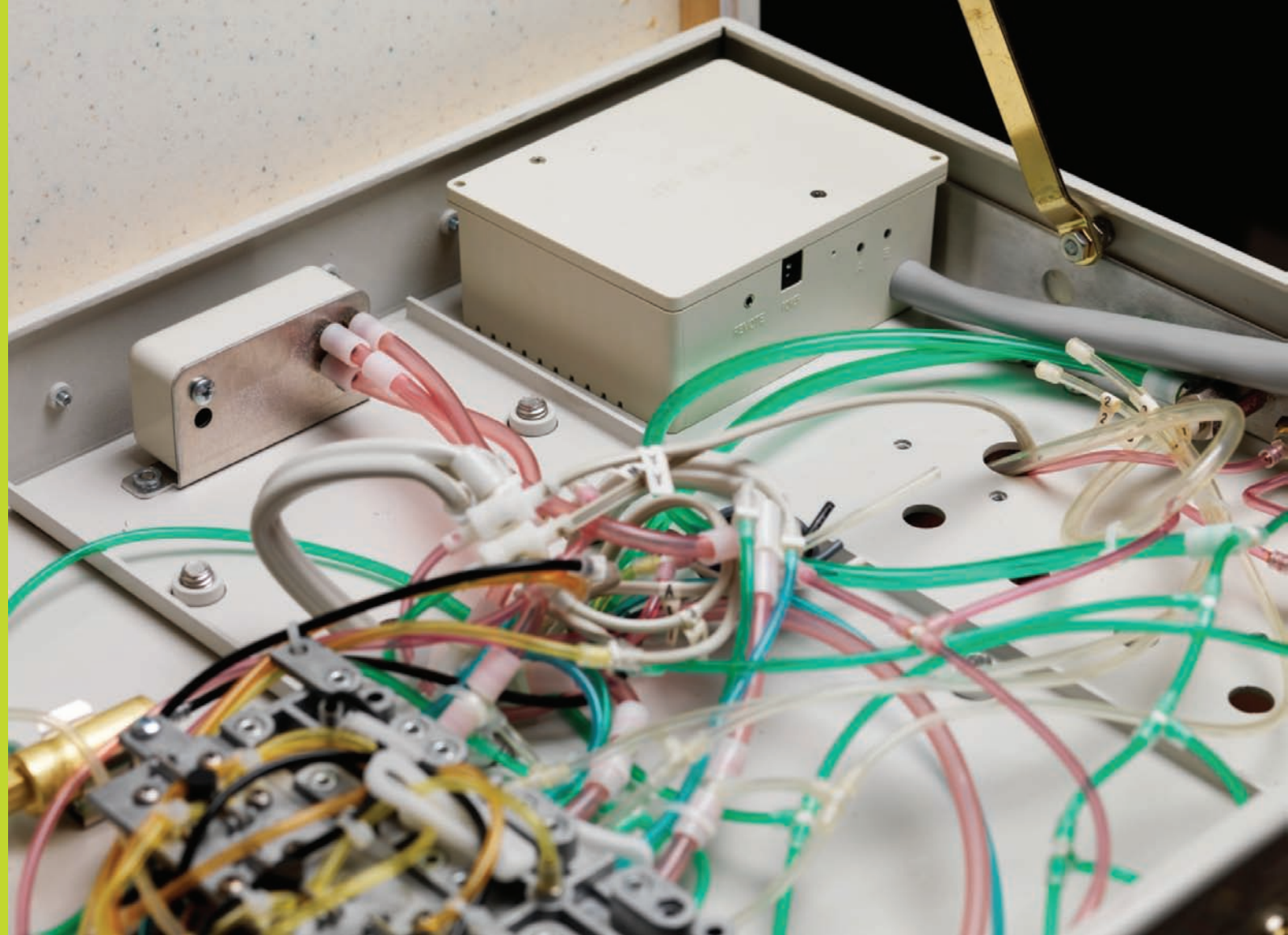

## DentalEZ® Galaxy®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the delivery head.

DentalEZ® Galaxy®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

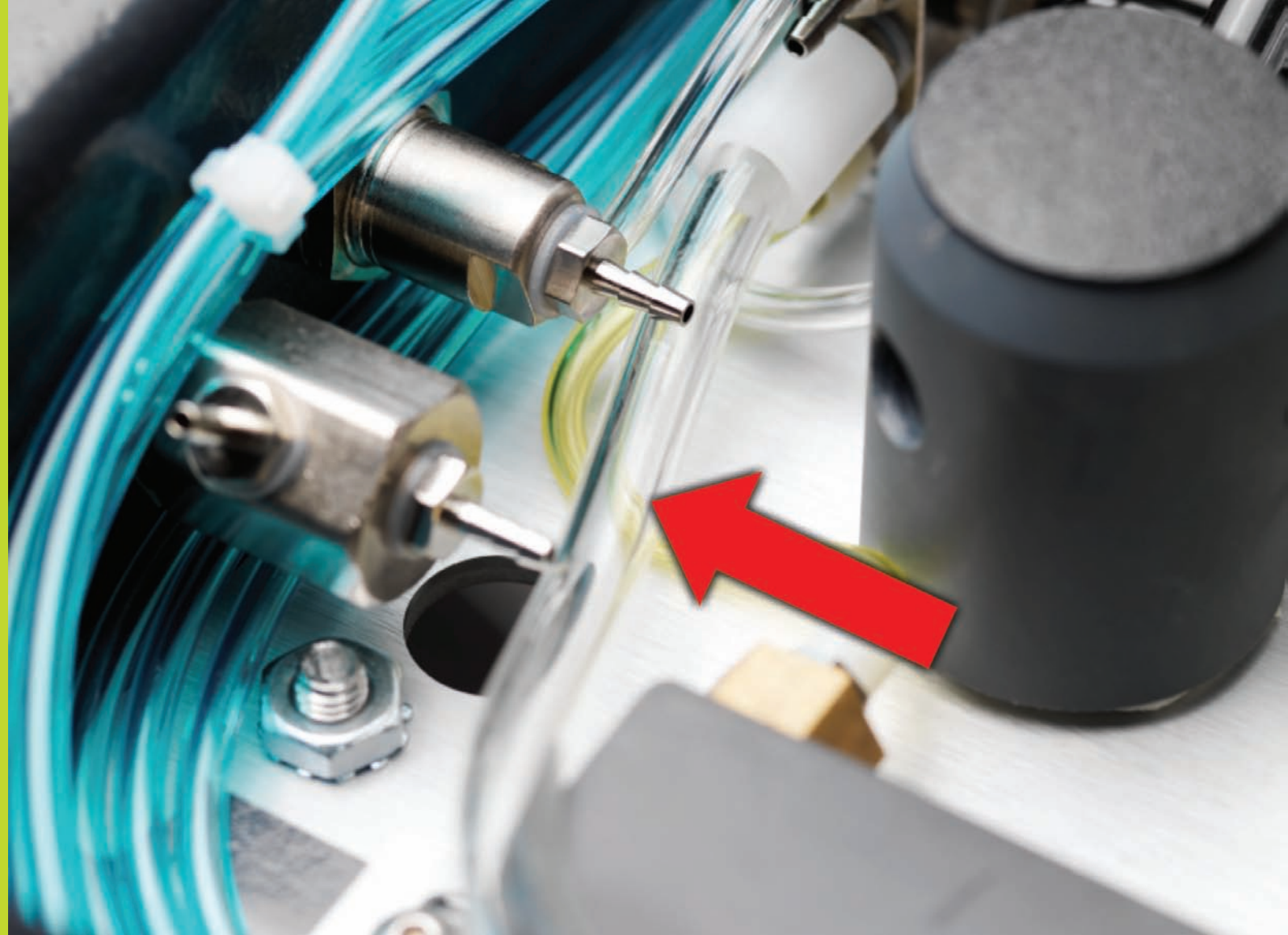

## Forest 4485

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Forest 4485

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Bolt the mounting bracket to the bottom of the delivery head and mount the controller to the bracket. This will allow you to overhang the unit.

The exhaust tube from the Handpiece Hose will not fit into the oil collector in the delivery head. Either open the hole up with a drill or feed the exhaust tube into the vent hole on the oil collector in the delivery head.

## Forest 4485

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Marus MAX100A

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

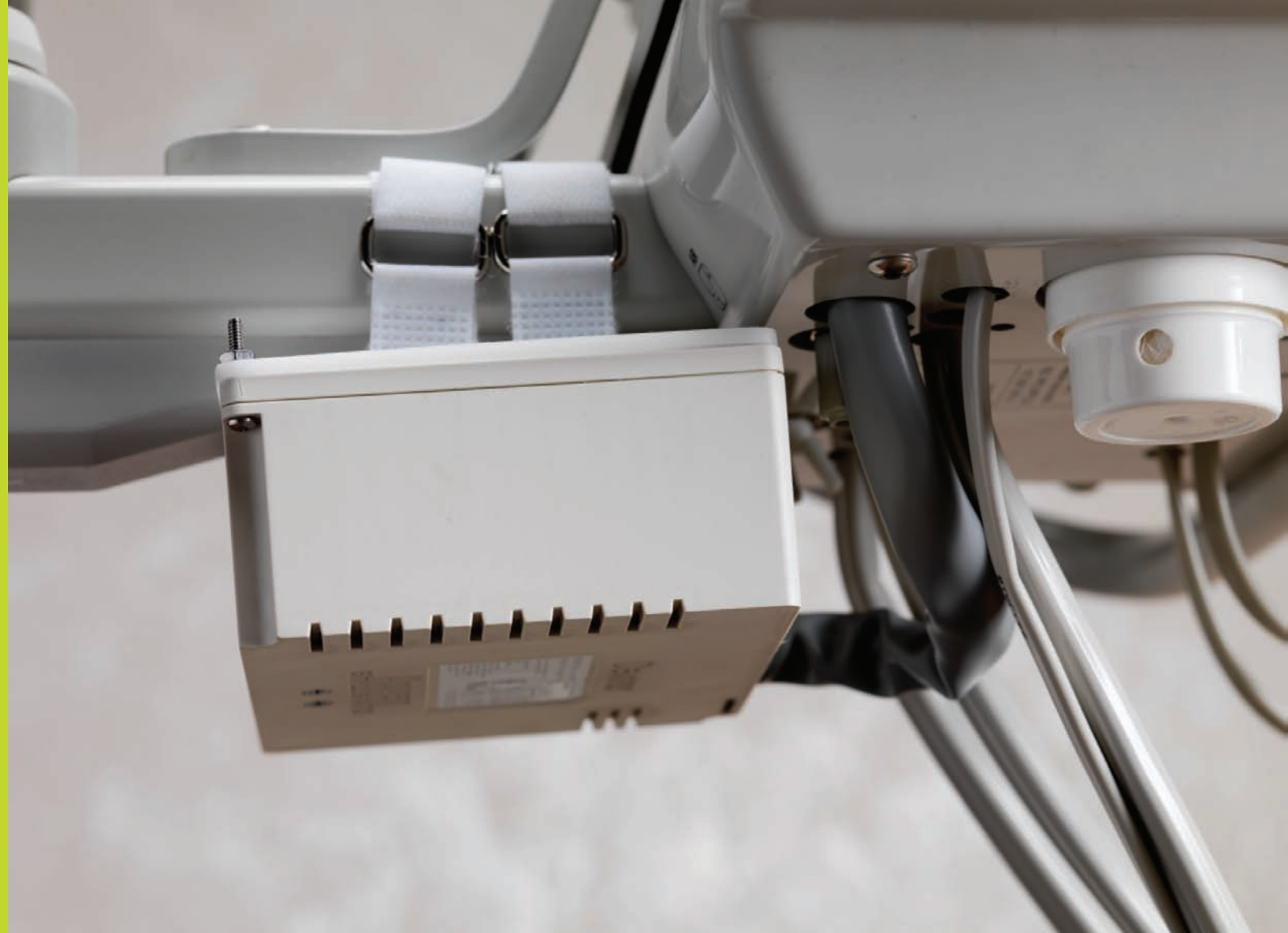

## Marus MAX100A

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Place the foam pad on the mounting bracket to prevent it from sliding around.

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the delivery head.

## Marus MAX100A

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

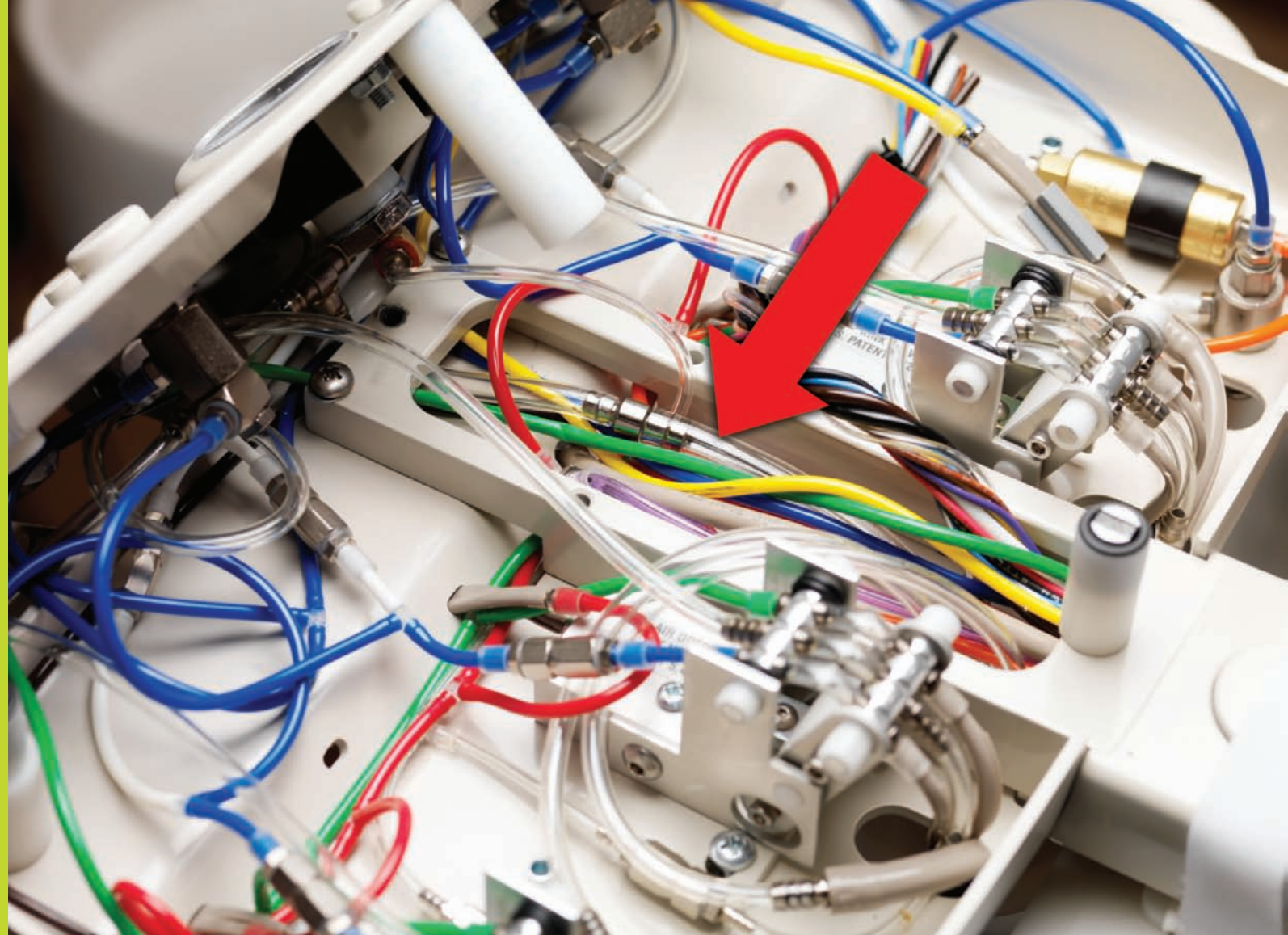

## Midmark 153600-8

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Midmark 153600-8

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Place the foam pad on the mounting bracket to prevent it from sliding around.

This unit has a pinch valve and will require use of a bleed fitting provided in the install kit on the yellow tube to the controller.

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the delivery head.

## Midmark 153600-8

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

See Marus MAX100A for details.

## Pelton and Crane FWS

Connecting the Controller Main Air | [Mounting the Controller](#) | [Installation Tips](#)

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®



## Pelton and Crane FWS

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Place the foam pad on the mounting bracket to prevent it from sliding around.

You will not need the oil cup provided in the starter kit. Connect to the Oil Collector in the delivery head.

## Pelton and Crane FWS

Connecting the Controller Main Air | Mounting the Controller | Installation Tips



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®


Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

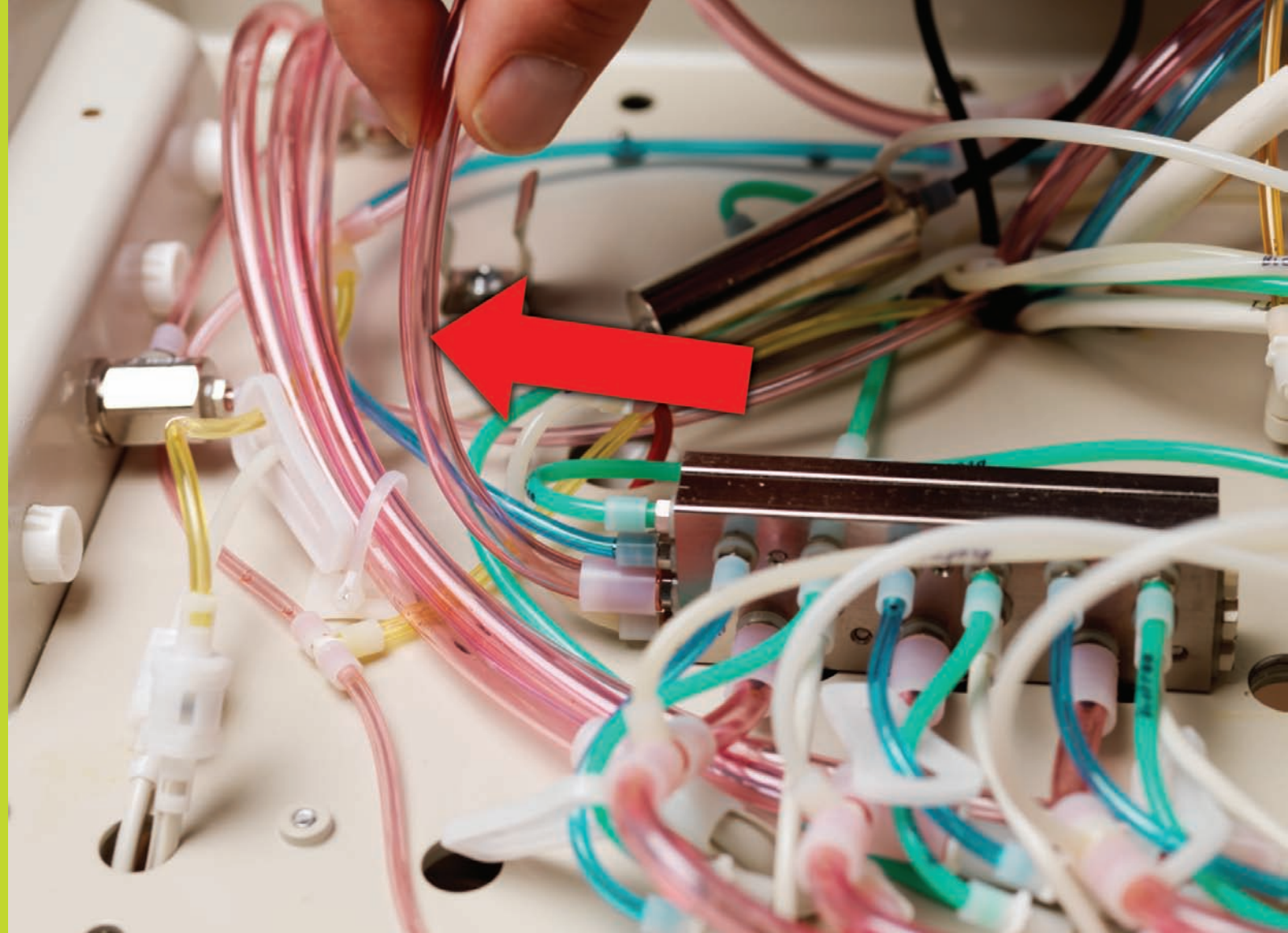

## Simplicity™ from DentalEZ®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)


**DENTSPLY**  
PROFESSIONAL



MIDWEST®

SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS

- ▼
- A-dec® 334
- A-dec® 532
- A-dec® 541
- A-dec® 542
- Belmont Quolis® 5000
- Biotec N7125
- DentalEZ® Galaxy®
- Forest 4485
- Marus MAX100A
- Midmark 153600-8
- Pelton and Crane FWS
- Simplicity™ from DentalEZ®

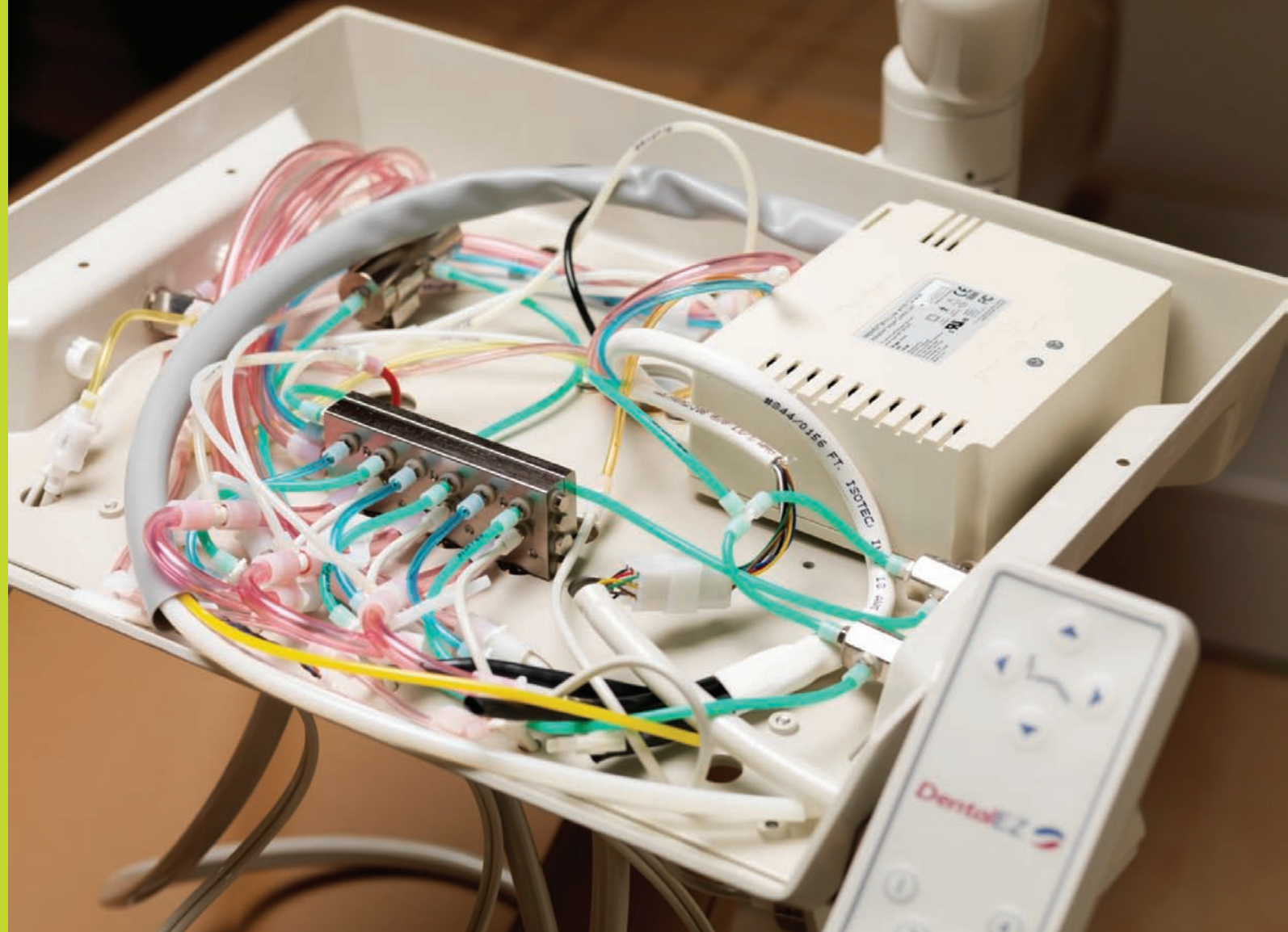

## Simplicity™ from DentalEZ®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.

MIDWEST®  
**STYLUS™ ATC**  
ANYTHING ELSE IS ORDINARY

[www.StylusATC.com](http://www.StylusATC.com)

**DENTSPLY**  
PROFESSIONAL



SELECT ONE OF THE  
FOLLOWING SYSTEMS FOR  
SPECIFIC INSTALLATION TIPS



A-dec® 334

A-dec® 532

A-dec® 541

A-dec® 542

Belmont Quolis® 5000

Biotec N7125

DentalEZ® Galaxy®

Forest 4485

Marus MAX100A

Midmark 153600-8

Pelton and Crane FWS

Simplicity™ from DentalEZ®

Some trays do not have an oil collector already installed. Use the one provided in the install kit in these cases.

## Simplicity™ from DentalEZ®

Connecting the Controller Main Air | Mounting the Controller | Installation Tips

©2010 DENTSPLY. All rights reserved.  
All brands are holders of their respective trademarks.